MED*intim*

personal healthcare

Caya[®] *contoured diaphragm*

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
Caya[®] *contoured diaphragm*

Instruction manual

Your **Caya[®]** diaphragm is a safe and easy-to-use contraceptive method. Always use **Caya[®]** in combination with a contraceptive gel (e. g. **Caya[®]** Gel). It is shaped to fit female anatomy. This shape and size were determined scientifically and fit most women.


Your **Caya[®]** diaphragm looks like this: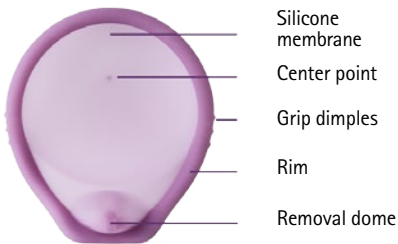

- **Silicone membrane:** completely covers the cervix and prevents sperm from entering the uterus
- **Center point:** palpable center point for better positioning of the diaphragm.
- **Grip dimples:** to facilitate vaginal insertion
- **Flexible rim:** to ensure correct position
- **Removal dome:** on the front of the diaphragm to facilitate an removal from the vagina

What is the position of the Caya[®] diaphragm in the vagina?

When inserted correctly, the diaphragm covers the cervix and is located between the recess on the pubic bone and the posterior vaginal walls.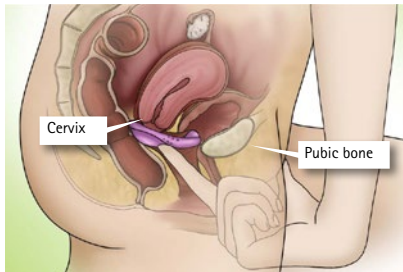

During and after sexual intercourse, the **Caya[®]** diaphragm prevents sperm cells from entering through the cervix into the uterus. Thus unwanted pregnancy is prevented.

NOTE: Please always use the diaphragm together with a contraceptive gel (e. g. **Caya[®]** Gel).


Practice inserting the **Caya[®]** diaphragm on your own a few times before using it for the first time during intercourse.

Before using Caya® the first time:

- Take your time
- Read the instruction manual carefully
- If possible, watch the DVD or the animation on the internet at www.caya.eu

- Find the cervix – it feels like the tip of your nose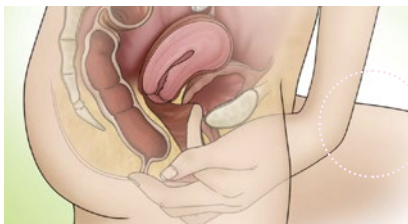

- Also find your pubic bone (in the vagina)

How do I insert the Caya® diaphragm?

NOTE: Insert the Caya® diaphragm before intercourse.

1. Remove the Caya® diaphragm from the package with clean hands and wash it with water and soap before using it the first time.
2. Before inserting the Caya® diaphragm prior to intercourse, place 4 ml (a teaspoon) of contraceptive gel (e. g. Caya® Gel) into the silicone membrane.

Spreading the gel around the rim facilitates insertion into the vagina.

Before inserting it into the vagina, fold the Caya® diaphragm using the grip dimples.

NOTE: If you have inserted the diaphragm more than 2 hours before sex, you need to add more contraceptive gel.


3. Choose a position which is comfortable for you – kneeling down, lying down or standing with one leg elevated

4. Hold the **Caya**® diaphragm with one hand. Place the thumb and index finger on the grip dimples along the rim. The arrow has to point towards your body.

5. Using your free hand, spread the labia. Push **Caya**® deeply along the posterior vaginal wall, towards your back, until the cervix rests inside the cup.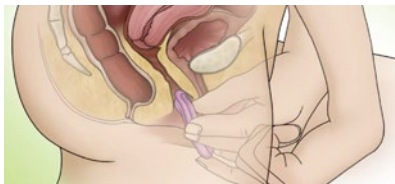


6. Use a finger to verify that the cervix is covered by the membrane of the **Caya**® diaphragm. The cervix must be completely covered. You should be able to feel it through the membrane. It feels similar to the tip of your nose.

If you cannot feel the cervix or the **Caya**®: Remove the diaphragm and try inserting it again.

Caya® is correctly positioned when the cervix is completely covered by the diaphragm and is between the posterior vaginal walls and the recess on the pubic bone.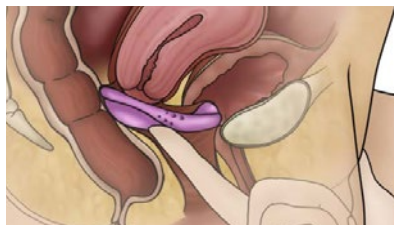


If you have verified both of the above and the diaphragm is in the correct position, safe contraception is ensured. You are now ready for intercourse.


7. After intercourse, the **Caya**® diaphragm must remain in the vagina for at least 6 hours. Be sure to never wear it longer than 24 hours without interruption.

NOTE: The **Caya**® diaphragm should never be left in the vagina for longer than 24 hours without interruption, as wearing it for longer and uninterrupted periods of time can promote bacterial growth inside the vagina. These bacteria can then lead to Toxic Shock Syndrome (TSS).

If intercourse is repeated or if the **Caya**® diaphragm was inserted more than 2 hours previously, additional gel should be applied to the diaphragm in place using an applicator; do not remove the diaphragm beforehand.

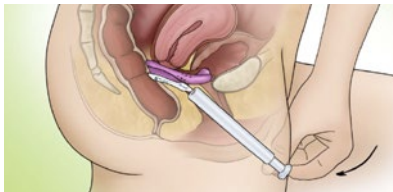
8. To remove the diaphragm, insert one finger into the vagina and pull out the **Caya**® diaphragm using the removal dome.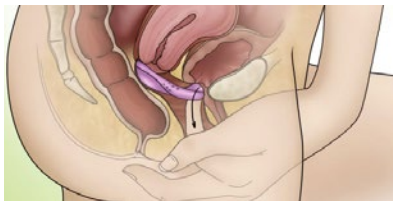

9. Cleaning and care of the **Caya**® diaphragm:

After removing the diaphragm, clean it with water and soap and allow it to air-dry naturally. Store the **Caya**® diaphragm in the original case. This ensures protection from external influences.

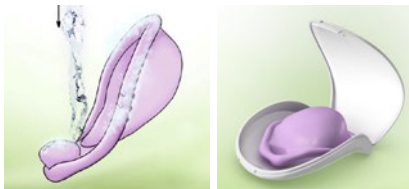The **Caya**® diaphragm is reusable. After the first use you can use it for two years.

10. Please check your **Caya**® diaphragm for holes and tears before each use. Make sure that you do not damage it with your fingernails.

When used in combination with a contraceptive gel, the **Caya**® diaphragm offers safe protection from unwanted pregnancy. However, no contraceptive method can guarantee 100% safety from unwanted pregnancy. The **Caya**® diaphragm does not protect you from sexually transmitted diseases such as e.g. HIV.

NOTE: Always keep the **Caya**® diaphragm in the original case and out of the reach of children.

When using, never forget:

- Never insert it longer than two hours before intercourse.
- **Always** use the **Caya®** diaphragm with a contraceptive gel.
- Check position by feeling whether the cervix is covered by the **Caya®** diaphragm.
- Wait to remove **Caya®** at least six hours after intercourse.
- Never leave in the vagina for longer than 24 hours without interruption.

Contraindication

- Use within the first 6 weeks after delivery/birth
- If a conventional diaphragm with a size of 85 mm and larger or 60 mm or smaller was previously used

Ask your healthcare provider, if:

- If you have difficulty getting the **Caya®** diaphragm to fit in your body comfortably (rare)
- You use treatments for vaginal infections (rare)
- You frequently suffer from urinary tract infections (rare)
- You have had lower abdominal/vaginal surgery (very rare)
- You have ever had a Toxic Shock Syndrome (very rare)

Please consult your doctor for any other medical questions.

NOTE: If you have forgotten to use the **Caya®** diaphragm and have had unprotected sexual intercourse, please contact your doctor immediately to discuss emergency contraception ("the morning-after pill").

Information

Tel: +49 6105 20 37 20
www.caya.eu

Technical data

Material:

Medical silicone, nylon

Contents:

- One **Caya®** diaphragm
- Instruction manual + DVD
- Case for storage

The shape and safety of the **Caya®** diaphragm were developed in a lengthy and intensive research process over many years in the USA and in worldwide industry-independent studies

NOTE: For production reasons, the silicone membrane may show slight variations in the colour gradient. This has no effect on the quality of the product or its safe usage.

FAQs

How safe is the Caya® diaphragm?

No contraceptive method can guarantee 100% safety from unwanted pregnancy. The **Caya®** diaphragm is just as safe as the classic diaphragm. The more often you use the **Caya®** diaphragm, the more confident you will become in using this contraceptive method and the more safety this method will offer you.

How do I know that Caya® is inserted correctly and fits?

Use your longest finger (usually the middle finger) to verify the correct position of the diaphragm: check whether the cup covers cervix and the removal dome is located securely behind your pubic bone. If this is the case you are well protected.

For how long can I use the Caya® diaphragm?

After the first use, the **Caya®** diaphragm should not be used for a period longer than 2 years.

Why does the Caya® diaphragm have to remain in the vagina for 6 hours after intercourse?

Sperm cells are no longer active after this period. These 6 hours are for your own safety and to prevent an unwanted pregnancy.

Why should Caya® not be worn for longer than 24 hours without interruption?

Wearing the Caya® diaphragm for longer than 24 hours without interruption can promote bacterial growth inside the vagina. These bacteria can lead to Toxic Shock Syndrome.

What should I do if I forget to remove Caya® after 24 hours have passed?

Remove Caya® and visit your doctor for an examination.

Can I have repeated consecutive intercourse with the Caya® diaphragm?

Yes, your Caya® will remain in position, but you will have to apply additional contraceptive gel with the aid of an applicator. Applicators for contraceptive gel can be purchased in the pharmacy.

Can the Caya® diaphragm also be used during menstruation?

The use of the diaphragm is possible.

What do I have to observe in case of a vaginal infection?

Please visit your doctor for treatment in case of a vaginal infection (e.g. fungus). Please use condoms during treatment and NOT a diaphragm. Upon completion of the treatment you should purchase and use a new Caya® diaphragm for your own safety.

Can I use lubricants together with the Caya® diaphragm?

The use of water-based lubricants (e. g. Sylik) has no influence on the use and safety of Caya®.

Could my partner feel the Caya® diaphragm?

This is highly unlikely. However, in rare cases, your partner may feel the rim of the diaphragm.

Does the Caya® diaphragm influence my sexual desire?

No, the Caya® diaphragm has no influence on your libido as it contains no hormones.

What should I do if Caya® causes pain during intercourse?

First verify the correct position of Caya® using a finger. If the Caya® diaphragm continues to cause pain, please visit your doctor.

Are there restrictions for the use of the Caya® diaphragm?

No, the only habit that requires changing is that one always uses the Caya® diaphragm when it is needed – during intercourse. The more often Caya® is used, the more confident one becomes in handling the diaphragm.

Can Caya® break or tear?

This is nearly ruled out. However, if you do notice that your Caya® is damaged in some way after intercourse, please visit your doctor immediately (within 5 days) to discuss the possibility of emergency contraception ("morning-after pill").

How can I positively influence safety when using the Caya® diaphragm?

The regular use and the use of a contraceptive gel (e.g. Caya® gel) are important measures for ensuring safety. In addition, the diaphragm should always be inserted before and not during intercourse.

What should I do if Caya® accidentally comes out during intercourse?

Re-insert Caya® again after applying some more contraceptive gel. If the diaphragm comes out after your partner has ejaculated or if you do not feel safe, visit your doctor to discuss whether you may need the "morning-after pill".

Can I additionally use lubricants with Caya®?

Yes, but only water-soluble lubricants. **Important:** Do not use any silicone oil lubricants. However, we cannot make any statements about the compatibility of the lubricants with the Caya® Gel or Contragel grün since we have no data available on this. That is, any limitations of the efficacy of the contraceptive gel cannot be ruled out. However, you can also use the Caya® Gel as a lubricant. It is made of lactic acid and cellulose and is free of parabens and benzalkonium chloride.

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KESSEL medintim GmbH